




INSTRUCTIONS FOR Auto Disable Syringes

INDICATIONS FOR USE

The auto disable syringe is intended for use in the suction and injection of vaccine for medical purposes. Additionally, after injection to the body, the plunger can be automatically locked by the mechanism to prevent the re-use of this syringe. Patient target group or groups: The auto disable syringes is suitable in each kinds of ages of patient.

MAIN STRUCTURE, PERFORMANCE CHARACTERISTICS AND EXPECTED CLINICAL BENEFITS

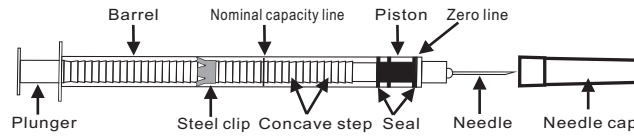


- Barrel is transparent, smooth, and can be observed liquid clearly.
- In the same plane, the plunger can be visible observed same-distance concave step, which set destructive area.
- Steel clip, between the barrel and plunger, is destructive after using.
- The syringe itself has fixed connected hypodermic needle.
- Expected clinical benefits: It minimizes the risk of cross infection.

SPECIFICATION: TKADS03: 0.1ml, 0.2ml, 0.25ml, 0.3ml, 0.4ml, 0.5ml, 1ml. Needle size: 23G to 31G.

USAGE

- 1) Peel open sterile wrapper.
- 2) Remove the protective cap without touching the needle.
- 3) Do not pull plunger backward before using the syringe.
- 4) Insert needle in clean rubber cap of vaccine vial and draw up a dose.
- 5) Expel air or excess vaccine, adjusting plunger to the indicated vaccine dose.
- 6) Inject vaccine.
- 7) Pull the syringe back.
- 8) Place the used syringe into the safety box (do not over-fill the safety box)

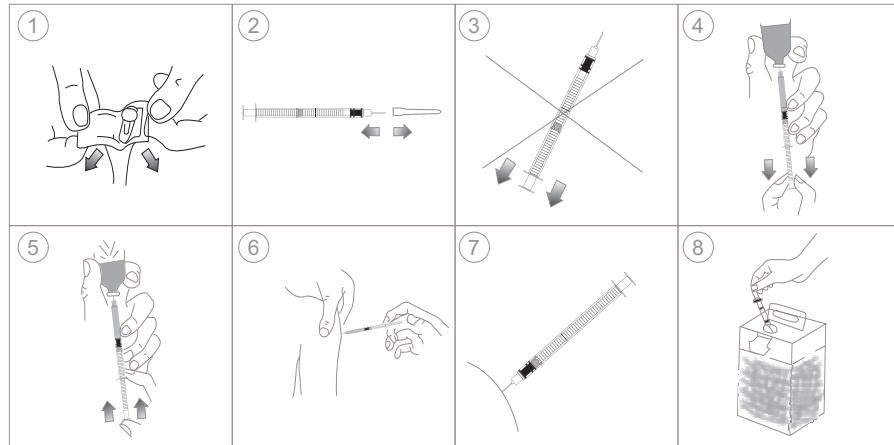

PRECAUTIONS

- The needle upwardly and expel all the air in syringe barrel, in case air be injected into human body.
- Abnormal operation, do not pull plunger. Sucking liquid, do not push back. because it will make the product destroyed.
- It should be sterile and non-pyrogenic before package opened.
- It has a valid period of five years. Do not use if it is beyond valid date.

WARNINGS

- Do not use if the package is opened or damaged, or protective cap is aparted or foreign matter is found.
- It should be used by special trained person, and the skin should be correctly sterilized.
- The products can not be reused, the repetition use may cause cross infection, even serious illnesses, such as hepatitis, HIV and other infectious disease.
- A notice to the user and/or patient that any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

STORAGE

-Product should be kept in a condition that relative humidity is less than 80%, it also should be free from corrosive gas, cool, dry with good ventilation and clean.

SYMBOLS

Unique device identifier	Batch code	Catalogue number	Caution	Sterilized using ethylene oxide
Date of manufacture	Manufacturer	Re-use prevention	Do not re-use	Do not use if package is damaged
Medical device	Use-by date	Do not resterilize	Fragile, handle with care	Temperature limit
Keep away from sunlight	Keep dry	Non-pyrogenic	This way up	Humidity limitation
				CE 2862

REV.: A/0 EFFECTIVE DATE: 2022-03-08

Anhui Tiankang Medical Technology Co., Ltd.
NO.228 Weiyi Road, Economic Development Zone, Tianchang City, Anhui, China

Shanghai International Holding Corp, GmbH (Europe)
Eiffestrasse 80 20537 Hamburg, GERMANY
Tel:0049-40-2513175