



INSTRUCTIONS FOR Auto Disable Syringes

INDICATIONS FOR USE

The auto disable syringes is specially designed as a tool for injection medicine to human body. The auto-disable feature automatically activates upon administration of the intended dose to prevent subsequent re-use of the syringe. Patient target group or groups: The auto disable syringes is suitable in each kinds of ages of patient.

Main structure, Performance characteristics and expected clinical benefits

- This product is made up of barrel, plunger, piston and needle.
- The syringe has auto-disable feature, and it will start when the plunger is pushed complete to the bottom of barrel.
- This barrel should be clean and transparent enough to easy observe.
- Barrel and piston matches well and it has a good property of sliding, can easy to use.
- Plunger is easy convenient for handle and usage.
- Expected clinical benefits: It minimizes the risk of cross infection.

SPECIFICATION

- TKADS01: 1ml, 2ml, 2.5ml, 3ml, 5ml, 10ml.
- TKADS02: 2ml, 2.5ml, 3ml, 5ml, 10ml.
- Needle size: 18G to 30G

USAGE

- First, Check valid period and completeness of package before using;
- Tear the package at the cut part and take out the syringe with needle;
- Push and pull plunger till it is well lubricated;
- Connect syringe with needle tight;
- Remove protective cap of needle, then do as sterile manipulation requirements.

PRECAUTIONS

- The needle upwardly and expel all the air in syringe barrel, in case air be injected into human body.
- It should be sterile and non-pyrogenic before package opened.
- It has a valid period of five years. Do not use if it is beyond valid date.

WARNINGS

- Do not use if the package is opened or damaged, or protective cap is aparted or foreign matter is found.
- It should be used by special trained person, and the skin should be correctly sterilized.
- The products can not be reused, the repetition use may cause cross infection, even serious illnesses, such as hepatitis, HIV and other infectious disease.
- A notice to the user and/or patient that any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

STORAGE

Product should be kept in a condition that relative humidity is less than 80%, it also should be free from corrosive gas, cool, dry with good ventilation and clean.

SYMBOLS

Unique device identifier	Batch code	Catalogue number	Caution	Sterilized using ethylene oxide	
Date of manufacture	Manufacturer	Re-use prevention	Do not re-use	Do not use if package is damaged	
Medical device	Use-by date	Do not re-sterilize	Fragile, handle with care	Temperature limit	CE 2862
Keep away from sunlight	Keep dry	Non-pyrogenic	This way up	Humidity limitation	

REV.: A/0 EFFECTIVE DATE: 2022-03-08

Anhui Tiankang Medical Technology Co., Ltd.
NO.228 Weiyi Road, Economic Development
Zone, Tianchang City, Anhui, China

Shanghai International Holding Corp. GmbH (Europe)
Eiffestrasse 80 20537 Hamburg, GERMANY
Tel:0049-40-2513175