




INSTRUCTIONS FOR Safety Insulin Syringes with Retractable Needles

受控

Application Scope

The product is just for injection insulin solution to human subcutaneous.

Product Performance Structure and Composition

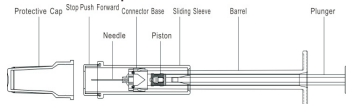

The product is made up of barrel, plunger, piston, protective cap, sliding sleeve, stop push forward and connector base.

Specification

- Syringe: 0.5ml, 1ml/U-40; 0.3ml, 0.5ml, 1ml/U-100.
- Needle: 25G~34G.

Usage

- First, check valid period and completeness of package before using;
- Tear the package at the cut part and take out the syringes;
- Remove the protective cap, pay attention to the protective cap shall not touch the tip, so as to avoid physical damage of the tip;
- Holding the syringe barrel, then push and pull plunger until it is well lubricated;
- Inhalation of insulin solution, then do as sterile manipulation requirements;
- After the injection, holding the syringe sliding sleeve and pulled up until the needle tip is completely covered in the sliding sleeve and "click" sounds then stop;
- Rotate the sliding sleeve until the "click" sound is locked and break the connection between the connector base and the barrel;
- Put into the specified box.

Contraindications and Precautions

- Remove the protective cap, pay attention to the protective cap shall not touch the tip, so as to avoid physical damage of the tip;
- Before the injection, first tip up, pushing the plunger, discharge the air in the barrel and needle, avoiding gas into the subcutaneous, forming mass;
- The product sterilized by EO. It should be sterile and free from pyrogen. Tear the package should be used immediately, only for single use;
- After use, this product should be discarded in the medical waste collection box, shall not be reused, and should be treated according to the relevant provisions of medical waste disposal;
- It has a valid period of five years, check valid period and completeness of package before using, do not use if it is beyond valid date;
- It should be used by special trained person and the skin should be correctly sterilized;
- Injection operation and monitoring should be carried out by professional medical staff, such as the product in the correct use of the process of accidents, should immediately stop the injection, and take corresponding emergency measures.

Warnings and Instructions

- Before use, check to ensure that the product is used insulin concentrations, the product can only the corresponding concentration of insulin injections;
- The product contains natural rubber, natural rubber allergy were prohibited to use;
- Do not use if the package is opened or damaged, or protective cap is aparted or foreign matter is found.

Storage

Product should be kept in a condition that relative humidity is less than 80%, temperature ranges from -10 to 40°C.

SYMBOLS

Unique device identifier	Batch code	Catalogue number	Caution	Sterilized using ethylene oxide
Date of manufacture	Manufacturer	Consult Instructions For Use	Do not re-use	Do not use if package is damaged
Medical device	Use-by date	Do not re-sterilize	Fragile, handle with care	Temperature limit
Keep away from sunlight	Keep dry	Non-pyrogenic	This way up	Humidity limitation
Single sterile barrier system	Sterile	REV.: B/0 EFFECTIVE DATE: 2025-05-30		

Anhui Tiankang Medical Technology Co., Ltd.
No. 228, Weiye Road, Economic Development
Zone, Tianchang City, Anhui Province, P.R. China.

EU REP Shanghai International Holding Corp. GmbH (Europe)
Eiffelstrasse 80 20537 Hamburg, Germany
Tel: 0049-40-2513175