



INSTRUCTIONS FOR DISPOSABLE HYPODERMIC NEEDLES

INDICATIONS FOR USE

Hypodermic needle for single use can be used in hypodermic injection. It suits users of different age and is the basic methods of infusion.

Main structure and properties

- The product is made up of, protective cap and needle, needle stand;
- This product has a good property of rigidity, toughness and corrosion resistance;
- Protective cap and needle seat match up well, and it will never loose by itself.

SPECIFICATION

18G to 30G

USAGE

- It is used together with other medical instrument;
- Check valid period and completeness of products before using;
- Open the package from the peel site and take out hypodermic needle;
- Connect it with syringe tightly;
- Pull out protective cap of needle. Then do as aseptic manipulation.

PRECAUTIONS

- Expel all air in barrel from the needle upwadslly, in case air be injected into human body;
- It should be sterile and free from pyrogen before package opened;
- It has a valid period of five years, do not use if it is beyond valid date;
- Discard after singe use.

WARNINGS

- Do not use if the package is opened or damaged, or protective cap is aparted, any part is missed, and foreign matter is found;
- It should be used by special trained person, and the skin should be correctly sterilized;
- The products can not be reused. The repetition use may cause cross infection, even serious illnesses such as hepatitis, HIV and other infectious disease.

STORAGE

Product should be kept in a condition that relative humidity is less than 80%, it also should be free from corrosive gas, cool, dry with good ventilation and clean.

Unique device identifier	Batch code	Catalogue number	Caution	Sterilized using ethylene oxide
Date of manufacture	Manufacturer	Consult Instructions For Use	Do not re-use	Do not use if package is damaged
Medical device	Use-by date	Do not re-sterilize	Fragile, handle with care	Temperature limit
Keep away from sunlight	Keep dry	Non-pyrogenic	This way up	Humidity limitation
Single sterile barrier system	STERILE Sterile	2862		
			REV.: C/0 EFFECTIVE DATE: 2025-05-30	