



INSTRUCTIONS FOR DISPOSABLE SYRINGES

INDICATIONS FOR USE

The disposable syringes is specially designed as a tool for vein and hypodermic injection solution to human body. The disposable syringes need to be used together with needle.

Main structure and properties

- This product is made up of barrel, plunger, piston.
- This barrel should be clean and transparent enough to easy observe.
- Barrel and piston matches well and it has a good property of sliding, can easy to use.
- Plunger is easy convenient for handle and usage.

SPECIFICATION

- Three-part syringes including 1ml to 60ml;
- Two-part syringes including 2ml to 20ml.

USAGE

- First, Check valid period and completeness of package before using;
- Tear the package at the cut part and take out the syringe without needle;
- Push and pull plunger till it is well lubricanted;
- Connect the syringe with needle, check that the needle is tightly connected to the syringe;
- Remove protective cap of needle, then do as sterile manipulation requirements.

Warnings & Precautions

- Expel all the air in barrel from the needle upwardly, in case gas boit come into being if air be injected into human body blood vessels or tumor come into being if gas be injected into subcutaneous;
- It should be sterile and free from pyrogen before package opened;
- It has a valid period of five years , do not use if it is beyond valid date;
- Discard after single use;
- Do not use if the package is opened or damaged, or protective cap is aparted or foreign matter is found;
- It should be used by special trained person, and the skin should be correctly sterilized;
- The products can not be reused. The repetition use may cause cross infection, even serious illnesses such as hepatitis, HIV and other infectious disease.

STORAGE

Product should be kept in a condition that relative humidity is less than 80%, it also should be free from corrosive gas, cool, dry with good ventilation and clean.

Unique device identifier	Batch code	Catalogue number	Caution	Sterilized using ethylene oxide	
Date of manufacture	Manufacturer	Consult Instructions For Use	Do not re-use	Do not use if package is damaged	
Medical device	Use-by date	Do not re-sterilize	Fragile, handle with care	Temperature limit	2862
Keep away from sunlight	Keep dry	Non-pyrogenic	This way up	Humidity limitation	
Single sterile barrier system		Sterile		REV.: B/0 EFFECTIVE DATE: 2025-05-30	

Anhui Tiankang Medical Technology Co., Ltd.
No. 228, Weiyei Road, Economic Development
Zone, Tianchang City, Anhui Province, P.R. China.

Shanghai International Holding Corp. GmbH(Europe)
Eiffestrasse 80 20537 Hamburg, Germany
Tel:0049-40-2513175